

Cleaning Agent Guide

Recommended Cleaning Solutions

Dräger knows the importance of hospital hygiene and infection control. We want to help keep your patients and workplace safe. Provided below, are cleaning agents that have been tested at our accredited Dräger Test Center and are compatible with all listed devices. There is no obligation to use these cleaning agents; however, Dräger strongly recommends the use of these approved cleaning agents on Dräger medical devices. Non-approved cleaning agents may cause harm and/or damage to the medical devices.

ANESTHESIA	Bruhin® BruTab 6S®	Clorox® Bleach Cleaner & Disinfectant Towels*	Ecolab® OxyCide™	Metrex® CaviWipes™
Apollo®	✓	✓	✓	✗
D-Vapor® 2000	✓	✓	✓	✗
D-Vapor® 3000	✓	✓	✓	✗
Fabius® GS Premium	✓	✓	✓	✗
Fabius® GS	✓	✓	✓	✗
Fabius® MRI	✓	✓	✓	✗
Fabius® Tiro	✓	✓	✓	✗
Fabius® Tiro M	✓	✓	✓	✗
MDS7®	✗	✓	✗	✓
Perseus® A500	✓	✓	✓	✗
Scio®	✓	✓	✓	✗
Vamos®	✓	✓	✓	✗
Vapor® 2000	✓	✓	✓	✗

PATIENT MONITORING	Bruhin® BruTab 6S®	Clorox® Bleach Cleaner & Disinfectant Towels*	Ecolab® OxyCide™	Metrex® CaviWipes™
IACS® with C500	✓	✓	✓	✗
IACS® with C700	✓	✓	✓	✗
Infinity® CentralStation	✓	✓	✓	✗
Infinity® Delta/Delta XL	✓	✓	✓	✗
Infinity® Gamma/XL/XXL	✓	✓	✓	✗
Infinity® Kappa	✓	✓	✓	✗
Infinity® Kappa XLT	✓	✓	✓	✗
Infinity® M300	✓	✓	✓	✗
Infinity® M540	✓	✓	✓	✗
Infinity® MultiView Workstation	✓	✓	✓	✗
Infinity® Omega	✓	✓	✓	✗
Infinity® Omega S	✓	✓	✓	✗
Infinity® SC6002 XL	✓	✓	✓	✗
Infinity® SC7000	✓	✓	✓	✗
Infinity® SC8000	✓	✓	✓	✗
Infinity® SC9000 XL	✓	✓	✓	✗
Infinity® TruST Telemetry	✓	✓	✓	✗

NEONATAL	Bruhin® BruTab 6S®	Clorox® Bleach Cleaner & Disinfectant Towels*	Ecolab® OxyCide™	Metrex® CaviWipes™
Babyleo® TN500	✓	✓	✓	✗
Babytherm® 8004/8010	✗	✗	✓	✗
BiliLux®	✗	✗	✓	✗
Caleo®	✗	✗	✓	✗
Globe-Trotter® GT5400	✓	✗	✓	✗
Isolette® 8000	✗	✗	✓	✓
Isolette® 8000 plus	✗	✗	✓	✗
Isolette® TI500	✗	✗	✓	✓
Jaundice Meter JM-103	✗	✗	✓	✓
Jaundice Meter JM-105	✗	✗	✓	✗
NanoBlu® 500	✗	✗	✓	✗
Photo-Therapy 4000	✗	✗	✓	✗
Resuscitaire®	✗	✗	✓	✓
TI500 Globe-Trotter®	✗	✗	✓	✓

RESPIRATORY	Bruhin® BruTab 6S®	Clorox® Bleach Cleaner & Disinfectant Towels*	Ecolab® OxyCide™	Metrex® CaviWipes™
Babylog® 8000 plus	✓	✓	✓	✗
Babylog® VN500	✓	✓	✓	✗
Carina®	✓	✓	✓	✗
Evita® 2 dura	✓	✓	✓	✗
Evita® 4	✓	✓	✓	✗
Evita® Infinity® V500	✓	✓	✓	✗
Evita® XL	✓	✓	✓	✗
Oxylog® 1000/2000/3000 (plus)	✓	✓	✓	✗

WORKPLACE INFRASTRUCTURE	Bruhin® BruTab 6S®	Clorox® Bleach Cleaner & Disinfectant Towels*	Ecolab® OxyCide™	Metrex® CaviWipes™
Agila®	✓	✓	✓	✗
Gemina® DUO	✓	✓	✓	✗
Movita®	✓	✓	✓	✗
Polaris® 50	✓	✓	✓	✗
Polaris® 100/200	✓	✓	✓	✗
Polaris® 600	✓	✓	✓	✗
Ponta®	✓	✓	✓	✗

*Clorox® Professional Disinfecting Bleach Cleaner & Dispatch® Hospital Cleaner Disinfectant Towels with Bleach
This guide does not replace or amend the instructions for use. It is always recommended to reference the latest instructions for use regarding the most current updates on approved disinfectants per device.